


# KOREAN lessons

SPA GARÇON 

Where every treatment becomes  
a ritual of rebirth and well-being

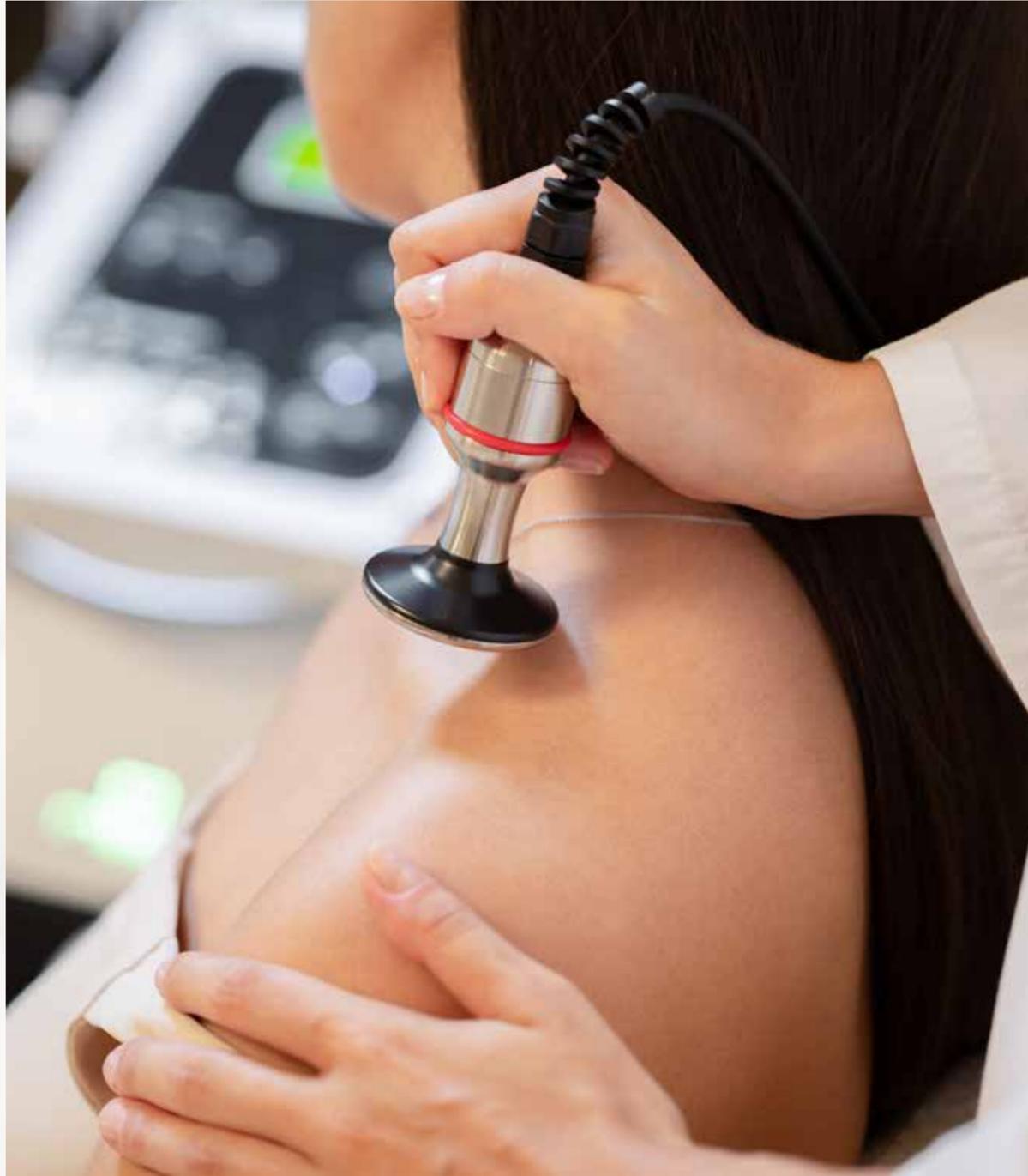

## SPA GARÇON

This multifunctional device transforms the world of hair care by combining the Italian tradition of wellness with Korean innovation. Spa Garçon revolutionizes the field of trichological care and takes the Head Spa concept to an unprecedented level.

### *Deep Scalp Cleansing*

The Aqua Peel handpiece, after product application, performs washing and suction using a lotion made of demineralized water and essential oils. This treatment removes impurities, excess sebum, and residue, ensuring deep cleansing and rebalancing of the scalp.



### *Galvanic Current*

Our device uses galvanic current to stimulate the epidermis and promote the deep penetration of specific treatments. The effectiveness of the products is enhanced, improving results on both the scalp and the hair. The inclusion of red-light heads allows for a more precise and gentle action, optimizing the controlled release of active ingredients.



### *Micro Bubble Shampoo*

The Spa Garçon is equipped with two tanks that generate a velvety foam inside, activating the surfactants so that the shampoo becomes gentler and more effective without irritating the scalp. The system also allows the product to be dispensed directly onto the hair, optimizing the professional's work during the massage.



### *The Integrated Airbrush*

The Hydro Spray precisely nebulizes ampoules and treatments, ensuring even distribution for optimal hydration and nourishment. This innovative approach not only enhances scalp health but also promotes hair growth and strengthening, helping to prevent hair loss and brittleness.



### *Micro Massage*

Technology at the service of the hairdresser: this device offers professionals in the sector a unique opportunity to evolve, combining the precision of cutting-edge technology with the value of manual skill and expertise.

A perfect combination to deliver personalized, effective, and avant-garde trichological treatments.



### *The Thermal Wellness Massage*

The Heating Massage head is the ideal ally to complete every treatment, designed to provide deep and targeted muscle relaxation. The result is an extraordinarily comfortable and sophisticated experience a true moment of self-care that enhances the overall sense of well-being, leaving the body relaxed and revitalized.



*A new standard in hair care that brings innovation to the world of trichology.*



Spa Garçon combines technologically advanced tools to:

- purify and strengthen the scalp,
- stimulate the epidermis and tone the scalp skin,
- promote the deep penetration of treatments.



# Benefits of the technologies

Innovation in the beauty industry is constantly evolving: the number of hair salons and wellness centers using new technologies aimed at promoting the health and well-being of the hair and scalp is steadily increasing.

Spa Garçon is the perfect addition to your salon because it:

- elevates the quality of the treatments offered,
- integrates traditional knowledge and treatments with technological innovation,
- expands the range of treatments and complementary products to be used in the salon or for at home maintenance.

The design and research behind Spa Garçon allow for conscious and personalized use: an experienced user, after proper training, can explore and evaluate how to create a fully customized treatment based on their expertise.

Even a beginner, thanks to the guided procedures and the protocols provided by Maletti, can confidently follow the training path to make the best use of the device and, over time, gain the necessary experience to take full advantage of all its features.



## SCIENCE AT YOUR FINGERTIPS

The studies carried out in collaboration with the University of Perugia have made it possible to incorporate into Spa Garçon the most innovative and advanced technologies on the market, carefully selected.



The key technologies of Spa Garçon are Peeling, Galvanic Current, and Photobiostimulation.

Spa Garçon is equipped with Aqua Peel, a system that allows the application of a solution onto the scalp while simultaneously suctioning away residues and impurities from the skin. The powerful suction of Aqua Peel enables deep cleansing without the need for abrasive

products, allowing its use directly at the workstation and enhancing both the client's comfort and overall experience.

Every skin type is different, and each person needs a fully customized treatment. Aqua Peel is a versatile tool: the professional can adjust the settings and regulate both the suction power and the solution flow, thereby optimizing treatment results.

The conscious choice of the solution allows for a completely personalized experience, selecting formulations tailored in active ingredients and fragrances, making each salon's touch unique and recognizable.

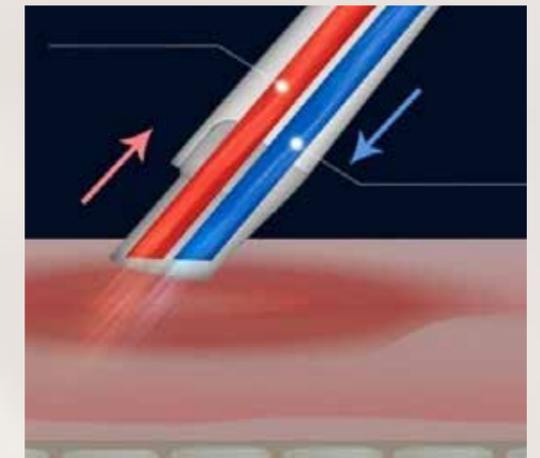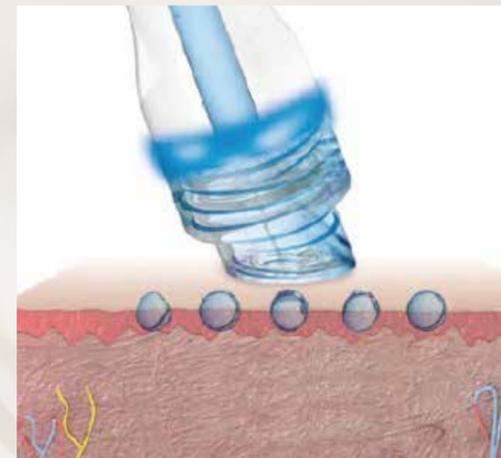

Analysis and Trial in collaboration with  
the Department of Medicine and Surgery, University of Perugia




Photobiostimulation is a non invasive technique that uses specific light sources to stimulate biological processes in tissues. Light has long been used in the medical field, as luminous radiation is absorbed in the form of photons by the tissues, promoting depending on the wavelength used specific beneficial or therapeutic effects at the tissue and cellular level. In the aesthetic field, light radiation with different wavelengths, usually between 450 and 650 nanometers, is used to interact with the most superficial layers of the skin.

Radiation at 450 nanometers produces more superficial effects, while that at 650 nanometers penetrates upto about 3 millimeters into the scalp. The radiation is absorbed by chromophores, specific areas of macromolecules present in the mitochondria, which trigger a series of chain reactions within the cell, improving its efficiency. The final result of this process is an enhancement of cellular respiration, the internal physiological functions of the cell, regenerative mechanisms, and local microcirculation.

## *BENEFITS OF PHOTOBIOSTIMULATION*

In collaboration with the University of Perugia, the Bionic Brush head was developed. Thanks to the combined action of Galvanic Current and LED light, it performs a synergistic action on the scalp, visibly improving its appearance and providing benefits to hair health. The violet, blue, and red LED lights of Spa Garçon enhance the effectiveness of the treatments: each wavelength penetrates to different depths, stimulating the scalp and facilitating the absorption of active ingredients. Moreover, the intensity of the Galvanic Current can be adjusted for a completely personalized treatment.

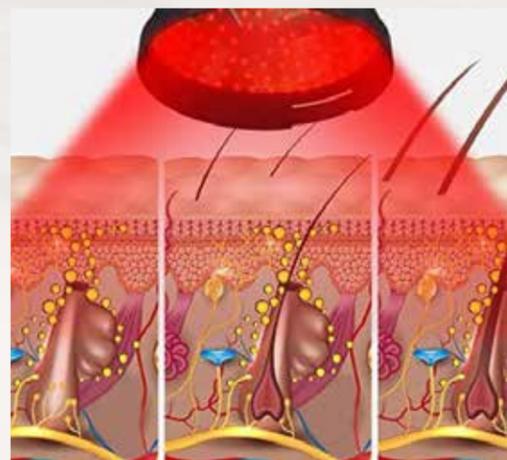

*Analysis and Trial in collaboration with  
the Department of Medicine and Surgery, University of Perugia*


## *BENEFITS OF GALVANIC CURRENT*

The main technology incorporated into Spa Garçon is galvanic current. Recent studies have shown that the application of galvanic current, even on the scalp, produces continuous beneficial effects that improve the appearance and health of the scalp and, consequently, of the hair.

Scientific literature confirms that the regular and prolonged use of this technology, combined with an appropriate selection of protocols and specific active ingredients,

generates localized effects on the skin that, in turn, enhance hair vitality and contribute to maintaining the overall health of both scalp and hair.

Spa Garçon allows professionals to design a fully personalized treatment thanks to the ability to adjust the intensity of the applied galvanic current and to use it in three different modes, optimizing and tailoring the experience to the fullest.

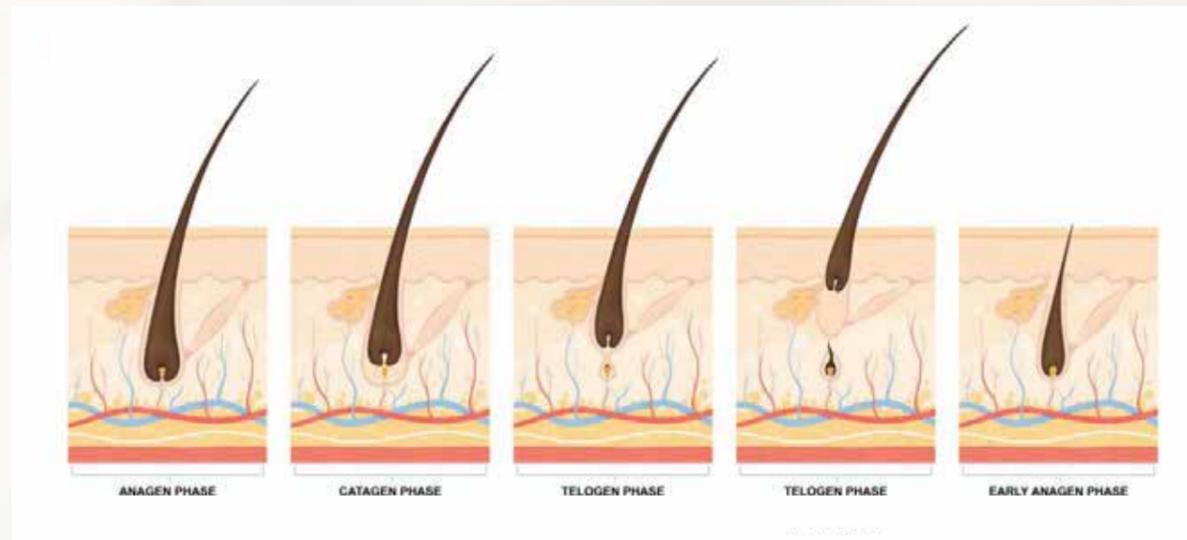

In the aesthetic and trichological fields, galvanic current is used in three possible modes:

- **SCALING** the current flows toward the biological tissue; electrons are drawn to the handpiece, producing a localized astringent effect and helping to detach impurities from the epidermis.
- **TONIC** electrons are released toward the biological tissue; the penetration of electrons promotes the absorption of active ingredients into the superficial layers of the skin.
- **MASSAGE** alternating emission of both currents to enhance local massage and achieve a combined effect.

The professional can select a specific protocol or design a set of personalized protocols adapted to the client's individual needs.



## *IN DETAIL*

Galvanic Current is a low-intensity direct current widely used in the aesthetic field due to its stability.

This technology makes it possible to achieve electrochemical, electrothermal, and electromechanical effects in biological tissues, which can be modulated according to the applied polarity and the current intensity.

Electrolytic-Chemical Effect:

- At the negative pole (cathode), there is an increase in pH due to the formation of hydroxides, producing a keratolytic, vasodilating, and stimulating effect.
- At the positive pole (anode), the pH decreases due to the formation of acids, resulting in an astringent, vasoconstrictive, and slightly antiseptic effect.

Electroosmotic Effect: promotes the movement of charged molecules and water within the tissues, supporting hydration and the transport of substances.

Vascular and Trophic Effect: prolonged stimulation induces an improvement in local microcirculation, with an increase in cellular metabolism and oxygenation.

Permeabilizing Effect: temporary increase in the permeability of the stratum corneum, useful for delivering active molecules deep into the skin (iontophoresis).



*Analysis and Trial in collaboration with  
the Department of Medicine and Surgery, University of Perugia*



*Protocols*



### *Welcome and consultation at the station*

Brief conversation with the client: ask about their needs, possible sensitivities, and desired hairstyle. Quickly perform a visual and tactile check of the scalp and hair fiber.

5 min



### *Application of scalp peeling*

Apply a restorative product or treatment, distributing it by hand or with a brush. Perform a relaxing scalp massage, ensuring the product is fully absorbed.

5 min



### *Use of Heating Massage*

Prepare the handpiece at the desired temperature (always check the temperature and pay attention to the client's sensitivity). Perform slow, circular movements on the neck, shoulders, temples, and hairline above the ears, avoiding excessive pressure. Focus on the neck and shoulders to release muscle tension and help the client relax more deeply.

5 min



### *Deep Scalp Cleansing*

Set the Aqua Peel handpiece according to the machine's instructions (suction intensity/solution flow). Perform systematic passes over the entire treatment area.

15 min



### *Micro Bubble Shampoo + specific mask*

Rinse thoroughly, apply the targeted shampoo, and repeat twice. Rinse again. Apply the specific mask, distributing it evenly. Comb if necessary and rinse thoroughly at the end.

10 min



### *Return to station + blow-dry service*

Apply the most suitable styling product according to the desired result and proceed with drying and styling.

Indicative time: about 30 minutes (short hair), up to 45 minutes (for complex styling or very long hair).

30-45 min

SUITABLE ALSO FOR TRAINEES  
HEATING MASSAGE | AQUA PEEL | MICRO BUBBLE SHAMPOO

H 1:10' / 1:25'



**Welcome and consultation at the workstation**

Brief interview with the client: ask about their needs, possible sensitivities, and desired hairstyle.

Quickly perform a visual and tactile assessment of the scalp and hair fiber.

10 min



**Application of scalp peeling**

Apply a restorative product or treatment, distributing it with the hands or a brush. Perform a relaxing scalp massage and ensure proper absorption of the product.

5 min



**Use of Heating Massage**

Prepare the handpiece at the desired temperature (always check the temperature and pay attention to the client's sensitivity). Perform slow, circular movements on the neck, shoulders, temples, and hairline above the ears to promote relaxation and muscle release.

5 min



**Deep Scalp Cleansing**

Set up the Aqua Peel handpiece according to the device instructions (suction intensity / solution flow). Perform systematic passes over the entire treatment area.

15 min



**Micro Bubble Shampoo**

Prepare the emulsion in the dispenser by mixing one part of the specific shampoo with four parts of water.

The foam that forms activates the surfactants in advance, making the product gentler and more effective on the scalp without causing irritation. Activate the handpiece and apply the shampoo directly at the workstation (the emulsion will be dense, allowing for optimal distribution). Have the client recline at the shampoo station and proceed with rinsing.

Apply a second dose of shampoo and, without rinsing, proceed with the Galvanic Y/T handpiece.

5 min



**Galvanic Current Y/T**

Set the Galvanic Y/T handpiece according to the technical guidelines. It helps improve follicular trophism, treats seborrheic scalp conditions, provides a keratolytic effect, and enhances shampoo cleansing. Perform systematic and controlled passes to stimulate microcirculation and prepare the scalp for the next phase.

5 min



**Rinsing and mask application**

Apply the specific mask, distributing it evenly. Comb if necessary and rinse thoroughly at the end.



**Hydro Spray Application**

Select the specific ampoule according to the scalp's needs (stimulating, soothing, balancing, etc.). Apply it using the Hydro Spray handpiece integrated into the device for more effective absorption.

5 min



**Bionic Brush Application**

Set the handpiece with galvanic current and select the appropriate LED light:

- Red: stimulating and revitalizing,
- Blue: purifying and soothing,
- Violet: balancing and regenerating.

Perform the passes for 5 minutes.

5 min



**Return to workstation + styling service**

Apply the most suitable styling product according to the desired result, then proceed with drying and styling.

Approximate duration: 30 min (short hairstyle) to 45 min (complex hairstyle or very long hair).

30-45 min

SUITABLE FOR SALON OWNERS AND SPECIALIZED OPERATORS  
HEATING MASSAGE | AQUA PEEL | GALVANIC Y/T  
MICRO BUBBLE SHAMPOO | HYDRO SPRAY | BIONIC BRUSH

H 1:25' / 1:40'



### *Welcome and consultation at the workstation*

Brief interview with the client: ask about their needs, possible sensitivities, and desired hairstyle. Quickly perform a visual and tactile assessment of the scalp and hair fiber.

5 min



### *Application of scalp peeling*

Apply the exfoliating or restorative product, distributing it with the hands or a brush. Perform a relaxing scalp massage and ensure proper absorption of the product.

5 min



### *Use of Heating Massage*

Prepare the handpiece at the desired temperature (always check the temperature and the client's sensitivity). Perform slow, circular movements on the neck, shoulders, temples, and hairline above the ears, avoiding excessive pressure. Focus on the neck and shoulders to relax the muscles and enhance the client's overall sense of well-being.

5 min



### *Deep Scalp Cleansing*

Set the Aqua Peel handpiece according to the device instructions (suction intensity / solution flow). Perform systematic passes over the entire treatment area for deep scalp cleansing.


12 min



### *Application of soothing product*

Apply the soothing product to the scalp and gently massage, distributing it evenly.

3 min

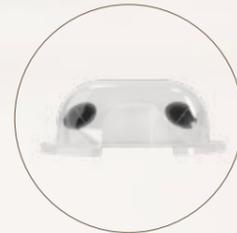

### *Client at the shampoo station + steam application*

Have the client sit comfortably at the shampoo station, adjusting the position and headrest correctly. Apply steam for 10 minutes to enhance the action of the soothing product or mask and to soften the hair fiber.

10 min



### *Micro Bubble Shampoo + specific mask*

Rinse thoroughly and apply the shampoo previously prepared in the dispenser. Allow the active foam to work on the scalp. Emulsify and rinse. Repeat twice and rinse again. Apply the specific mask, distributing it evenly; comb if necessary, and rinse at the end. Optionally, the Shirodara cascade can be used to promote client relaxation and well-being.

10 min



### *Return to workstation + styling service*

Apply the most suitable styling product according to the desired result, then proceed with drying and styling. Approximate duration: 30 min (short hairstyle) to 45 min (complex hairstyle or very long hair).

30-45 min

ALSO SUITABLE FOR TRAINEES  
HEATING MASSAGE | AQUA PEEL  
STEAM | MICRO BUBBLE SHAMPOO | SHIRODARA CASCADE

H 1:20' / 1:35'



### *Welcome and consultation at the workstation*

Brief interview with the client: ask about their needs, possible sensitivities, and desired hairstyle. Quickly perform a visual and tactile assessment of the scalp and hair fiber.

10 min



### *Application of scalp peeling*

Apply the exfoliating or restorative product, distributing it with the hands or a brush. Perform a relaxing scalp massage and ensure proper absorption of the product.

5 min



### *Use of Heating Massage*

Prepare the handpiece at the desired temperature (always check the temperature and the client's sensitivity). Perform slow, circular movements on the neck, shoulders, temples, and hairline above the ears, avoiding excessive pressure. Focus on the neck and shoulders to relax the muscles and enhance the client's sense of well-being.

5 min



### *Deep Scalp Cleansing*

Set the Aqua Peel handpiece according to the device instructions (suction intensity / solution flow). Perform systematic passes over the entire treatment area for deep scalp cleansing.

12 min



### *Application of soothing product*

Apply the soothing product to the scalp and gently massage to distribute it evenly.

3 min



### *Client at the shampoo station + steam application*

Have the client sit comfortably at the shampoo station, adjusting the position and headrest correctly. Apply steam for 10 minutes to enhance the action of the soothing product or mask and to soften the hair fiber.

10 min



### *Micro Bubble Shampoo + specific mask*

Rinse thoroughly and apply the specific shampoo previously prepared in the dispenser: one part shampoo to four parts water (double wash with active foam). This process activates the surfactants before application, making the shampoo gentler and more effective without irritating the scalp. Rinse. Apply the specific mask, distributing it evenly, and comb if necessary. After the exposure time, rinse again.

10 min



### *Shirodara Cascade*

Activate the Shirodara cascade. It promotes scalp relaxation, deep rest, and stimulates microcirculation.

5 min



### *Hydro Spray Application*

Select the specific ampoule according to the scalp's needs (stimulating, soothing, balancing, etc.). Apply it using the Hydro Spray handpiece integrated into the device for more effective absorption.

5 min



### *Bionic Brush Application*

Set the handpiece with galvanic current and select the appropriate LED light:

- Red: stimulating and revitalizing,
- Blue: purifying and soothing,
- Violet: balancing and regenerating.

Perform the passes for 5 minutes.

5 min



### *Return to workstation + styling service*

Apply the most suitable styling product according to the desired result, then proceed with drying and styling.

Approximate duration: 30 min (short hairstyle) to 45 min (complex hairstyle or very long hair).

30-45 min

SUITABLE FOR SPECIALIZED OPERATORS  
HEATING MASSAGE | AQUA PEEL | STEAM | MICRO BUBBLE SHAMPOO  
SHIRODARA CASCADE | HYDRO SPRAY | BIONIC BRUSH

H 1:40' / 1:55'



### *Welcome and consultation with trichocamera*

Brief interview with the client: ask about their needs, possible sensitivities, and desired hairstyle.

Perform a detailed analysis of the scalp and hair fiber using the trichocamera for precise observation.

10 min



### *Application of scalp peeling*

Apply the exfoliating or restorative product, distributing it with the hands or a brush. Perform a relaxing scalp massage and ensure proper absorption of the product.

5 min



### *Use of Heating Massage*

Prepare the handpiece at the desired temperature (always check the temperature and the client's sensitivity). Perform slow, circular movements on the neck, shoulders, temples, and hairline above the ears, avoiding excessive pressure.

Focus on the neck and shoulders to relax the muscles and enhance the client's overall sense of well-being.

5 min



### *Deep Scalp Cleansing*

Set the Aqua Peel handpiece according to the device instructions (suction intensity / solution flow). Perform systematic passes over the entire treatment area for a deep scalp cleansing.


12 min



### *Application of soothing product*

Apply the soothing product to the scalp following the technical instructions. Gently massage to distribute it evenly.


3 min

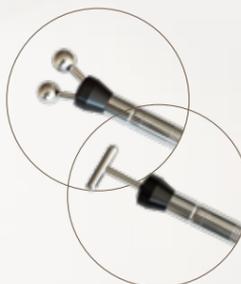

### *Galvanic Current Y/T*

Set the Galvanic Y/T device according to the manual. The increase in local blood flow and lymphatic drainage allows for better penetration of the active ingredients in the chosen product. Perform systematic and controlled passes to stimulate microcirculation and prepare the scalp for the next phase.

5 min

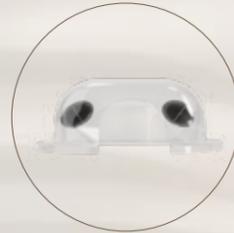

### *Client at the shampoo station + steam application*

Have the client sit comfortably at the shampoo station, adjusting the position and headrest correctly. Apply steam for 10 minutes to enhance product action and soften the hair fiber.

10 min



### *Micro Bubble Shampoo + specific mask*

Rinse thoroughly and apply the specific shampoo previously prepared in the dispenser: one part shampoo to four parts water (double wash with active foam). This process activates the surfactants before application, making the shampoo gentler and more effective without irritating the scalp. Rinse. Apply the specific mask, distributing it evenly, and comb if necessary. After the exposure time, rinse again.

10 min



### *Shirodara Cascade*

Activate the Shirodara cascade. It promotes scalp relaxation, deep rest, and stimulates microcirculation.

5 min



### *Hydro Spray Application*

Select the specific ampoule according to the scalp's needs (stimulating, soothing, balancing, etc.). Apply it using the Hydro Spray handpiece integrated into the device for more effective absorption.

5 min



### *Bionic Brush Application*

Set the handpiece with galvanic current and select the appropriate LED light:

- Red: stimulating and revitalizing,
- Blue: purifying and soothing,
- Violet: balancing and regenerating.

Perform the passes for 5 minutes.

5 min



### *Return to workstation + styling service*

Apply the most suitable styling product according to the desired result, then proceed with drying and styling.

Approximate duration: 30 min (short hairstyle) to 45 min (complex hairstyle or very long hair).

30-45 min

SUITABLE FOR SALON OWNERS AND SPECIALIZED OPERATORS  
HEATING MASSAGE | AQUA PEEL | STEAM | MICRO BUBBLE SHAMPOO  
SHIRODARA CASCADE | HYDRO SPRAY | BIONIC BRUSH

H 1:45' / 2:00'



Piazzale Guerrino Maletti, 1 42019 Scandiano (RE) - Italy  
tel +39 0522 7631, info@malettigroup.com www.malettigroup.com